LC-Primer

Processing Information

Description/ LC- Primer is a light-curing primer for a chemical compound between PET-G pressure indication:

moulding material and DURASPLINT® LC and CLEAR-BLOKKER®.

It can also be used for the compound of already polymerised DURASPLINT® LC

with CLEAR-BLOKKER® in case of build-ups or corrections.

Composition: Methyl methacrylate, acrylic acide ester, photoinitiator

Curing: 5 minutes in light-curing oven (LC-6 Light oven) equipped with 3 UV-A tubes

and 3 VIS tubes (wavelength range 350 – 450 nm), alternating order of tubes.

Packaging unit: 10 ml

Storage condition: 4 – 25°C, do not expose to direct sunlight.

 $C \in$



SCHEU-DENTAL GmbH www.scheu-dental.com Am Burgberg 20 58642 Iserlohn · Germany phone +49 2374 9288-0 +49 2374 9288-90 Facebook scheu dental



