Laborinformation

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Laboratory information

SCHEU-DENTAL pressure moulding machines:

Please place the BRUX CHECKER® foil (without white separating foil) on the O-ring of the adjustment ring so that the imprint on the foil is legible. Secure the foil by means of the locking ring. Place the model in the pellets or on the model platform so that the dental arch can be easily moulded. You can achieve optimal results by embedding the model up to 5 mm below the gingival margin. Ensure that the model is always placed with the incisors facing towards the locking handle.

Machines of other manufacturers:

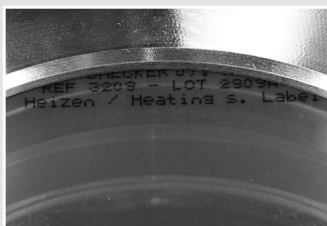
Foils of 120 mm diameter carry the imprint „coated side“. Make sure that after thermoforming the imprint on the model can be read properly. The right application can be verified by a scratch test.

Safety note:

Only use in well-ventilated rooms, as vapours with a characteristic pungent odour may be produced during processing. Serious incidents in connection with the use of the product shall be reported immediately to the manufacturer and the competent authority.

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Patient information

Your dentist has given you one or more BRUX CHECKER® foils which were moulded over your maxillary cast.

A biocompatible colour coat has been applied to one side of this foil, which will be grinded on the contact points between upper and lower jaw. By means of the grinding pattern which will be thus received, your dentist can evaluate potential disorders of your stomatognathic system such as grinding (bruxism).

In order to obtain a meaningful result, please observe the following:

- Make sure not to use any mouthwash containing alcohol, as this might dissolve the lacquer coat of the splint and lead to a discoloration of the tongue.
- The foil will be worn during sleep in the night.
- Carefully press the foil on your upper and lower jaw, first in the area of the front teeth and then in the area of the molars.
- Please proceed in reverse order in the morning: Remove the foil using your finger nails first in the area of the molars and then in the area of the front teeth.
- Rinse the foil under lukewarm water and let it dry. Then put the foil again over the plaster model, in order to avoid deformation.
- Each foil is for use during one night only.
- Please avoid alcohol and medication like sleeping pills, as this may lead to a non-typical grinding pattern.
- Once you returned the model and the foil to your dentist, he will consult you on the next measures to take.

Note: Allergic reactions to components of BRUX CHECKER® (e.g. erythrosine (E127)) cannot be ruled out in sensitised persons.