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EN Instructions for use

Intended purpose: Thermoforming material for the fabrication of appliances and splints for dental and orthodontic treatment

Indication: Suitable only for the fabrication of aligner splints, OSAMU-Retainers®, retainers, retention splints, permanent retainers and bite splints. Instructions for use can be found at www.scheu-dental.com/pressure-moulding-material.

Contraindication: Allergic reactions to components of IMPRELON® S/IMPRELON® S+ (copolyester) cannot be ruled out in sensitised people. Any use not stated above is contraindicated.

Storage: Store at 5°C to 35°C. Protect from sunlight. Keep dry.

Important: Must only be used by qualified personnel and within 15 minutes after opening. Only open the barrier bag if it is undamaged and still sealed. For single use. Not suitable for build-up with resins. Any serious incident involving the product must be reported to the manufacturer and the competent authority immediately.

Safety information: The practitioner is responsible for correct use. The splint must not be used if the material is cracked/broken/contains bubbles; there is a risk of swallowing or aspiration. When using IMPRELON® S with 3D-printed models, a non-perforated protective foil (ISOFOLAN®) or alternatively IMPRELON® S+ must be used. Remove the protective foil after the thermoforming process. Do not bring the products into contact with alcoholic/organic solvents.

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